


90° DISS Style Outlets - PX-XF Series

Oxygen Outlet Shown

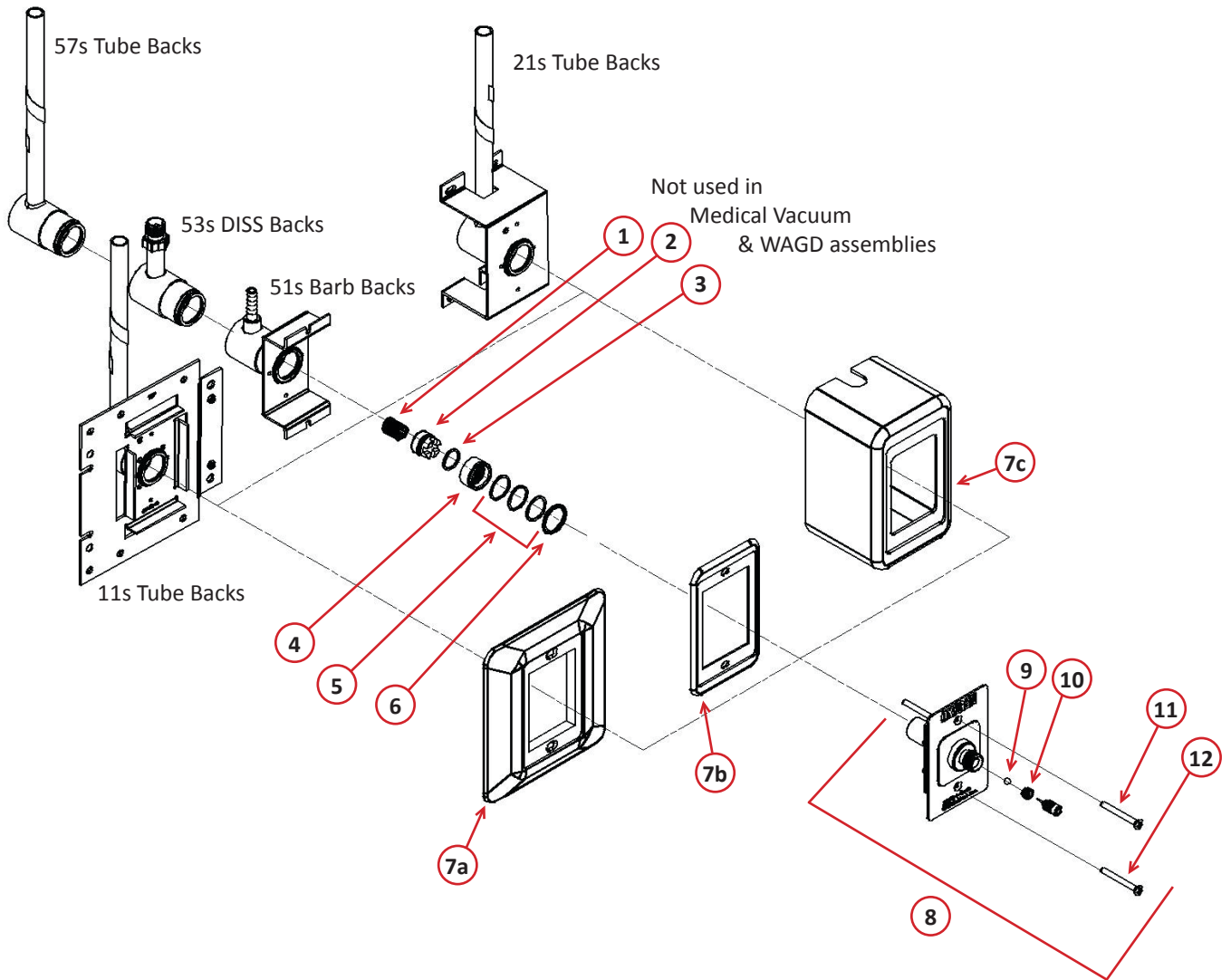

Item 8 includes items 9-12 in the Finishing Kit

Bill of Material

Item	Part Number	Description	Quantity	Note
1	PX-13-1003R	Spring	1	
2	PX-XC1000-5R	Poppet	1	
3	PX-11-0050R	Poppet O-Ring	1	A
4	PX-XC1000-6R	Insert Sleeve	1	
5	PX-11-0047R	Sleeve O-Ring	3	B
6	PX-12-1045R	Retaining Ring	1	
7a	PX-XC1000-9R	Finishing Kit Trim Plate	1	
7b	PX-XC1000-10R	Trim Plate Cover	1	
7c	PX-XC2100-3R	Finishing Kit Housing (Surface Mount)	1	
8	See Below	XF Finishing Kit Assemblies	See Below	
9	PX-F1000-13R	Ball Check	1	
10	PX-F1000-145R	DISS CV Seat	1	
11	See Below	Cartridge Assy (Primary Valve Kit)	See Below	
12	PX-30-0003	Mounting Screw Kit (2) PX-12-3037	1	
Item 8 Outlet Finishing Kits				
(all kits are assembled with items 9-12 except Medical Vacuum and WAGD, items 9 and 10 are not required)				
	PX-XF1004-A	Nitrous Oxide	1	
	PX-XF1008-A	Carbon Dioxide	1	
	PX-XF1012-A	Nitrogen	1	
	PX-XF1016-A	Medical Air	1	
	PX-XF1022-A	Medical Vacuum	1	
	PX-XF1024-A	Oxygen	1	
	PX-XF1032-A	WAGD (Waste Anesthetic Gas Disposal)	1	
	PX-XF102080-A	Instrument Air	1	
Item 11 Cartridge Assembly (Primary Valve Kit)				
	PX-F1004RC	Nitrous Oxide	1	
	PX-F1004RC	Carbon Dioxide	1	
	PX-F1012RC	Nitrogen	1	
	PX-F1000RC	Medical Air	1	
	PX-F1022RC	Medical Vacuum	1	
	PX-F1024ARC	Oxygen	1	
	PX-F1032RC	WAGD	1	
	PX-F102080RC	Instrument Air	1	
Secondary Valve Kit				
	PX-XC1000KR	Secondary Valve Kit for all Gases Except Medical Vacuum & WAGD (Kit Includes Items 1 through 6)	1	
	PX-XC1022KR	Secondary Valve Kit for Medical Vacuum & WAGD (Kit Includes Items 4 through 6)	1	

NOTE: A – Install O-Ring item 3 on Poppet item 2
 B – Install O-Rings item 5 onto Insert Sleeve item 4