


180° Allied/Chemetron Style Outlets - PX-XA Series

Oxygen Outlet Shown

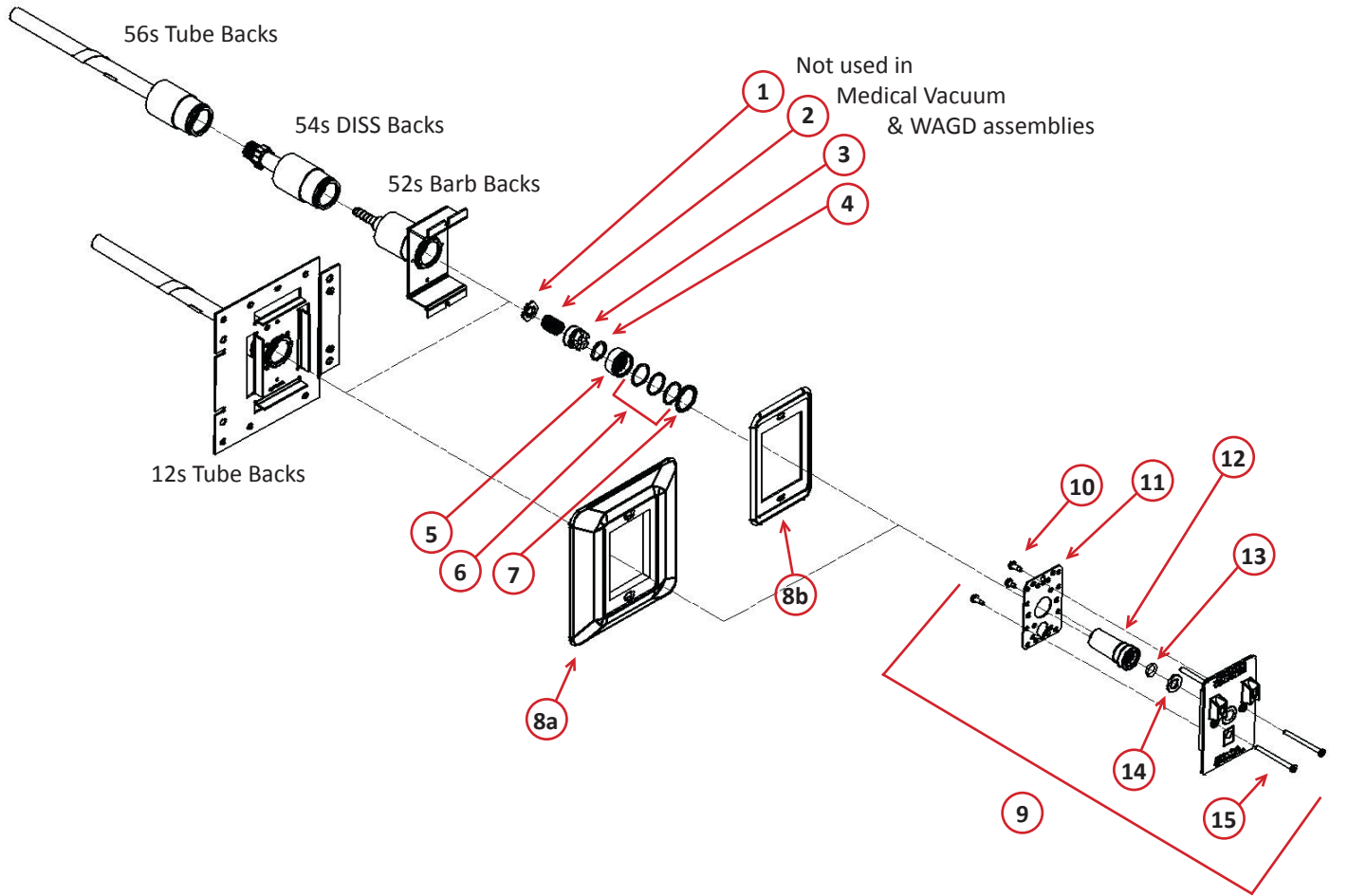

Item 9 includes items 10-15 in the Finishing Kit

Bill of Material

Item	Part Number	Description	Quantity	Note
1	PX-XC1000-7R	Spacer Washer	1	
2	PX-13-1003R	Spring	1	
3	PX-XC1000-5R	Poppet	1	
4	PX-11-0050R	Poppet O-Ring	1	A
5	PX-XC1000-6R	Insert Sleeve	1	
6	PX-11-0047R	Sleeve O-Ring	3	B
7	PX-12-1045R	Retaining Ring	1	
8a	PX-XC1000-9DCR	Finishing Kit Trim Plate	1	
8b	PX-XC1000-10R	Trim Plate Cover	1	
9	See Below	XA Finishing Kit Assemblies	See Below	
10	PX-12-3084R	Mounting Screws	3	
11	PX-AQ1000-12R	Indexing Housing Cover	1	
12	PX-XA1000-DR	Front Body Assembly	1	
13	PX-11-0005R	Front Body O-Ring	1	C
14	PX-12-1002R	Plated Washer	1	
15	PX-30-0003	Mounting Screw Kit (2) 12-3037	1	
Item 9 Outlet Finishing Kits (all kits are assembled with items 10-15)				
	PX-XA1004-A	Nitrous Oxide	1	
	PX-XA1008-A	Carbon Dioxide	1	
	PX-XA1016-A	Medical Air	1	
	PX-XA1022-A	Medical Vacuum	1	
	PX-XA1024-A	Oxygen	1	
	PX-XA1032-A	WAGD (Waste Anesthetic Gas Disposal)	1	
Secondary Valve Kit				
	PX-XC1000KR	Secondary Valve Kit for all Gases Except Medical Vacuum & WAGD (Kit Includes Items 1 through 7)	1	
	PX-XC1022KR	Secondary Valve Kit for Medical Vacuum & WAGD (Kit Includes Items 5 through 7)	1	

NOTE: A – Install O-Ring item 4 on Poppet item 3
 B – Install O-Rings item 6 onto Insert Sleeve item 5
 C – Install O-Ring item 13 onto Front Body Assembly item 12 and cover with Washer item 14