


90° Puritan Bennett Style Outlets - PX-XP Series

Oxygen Outlet Shown

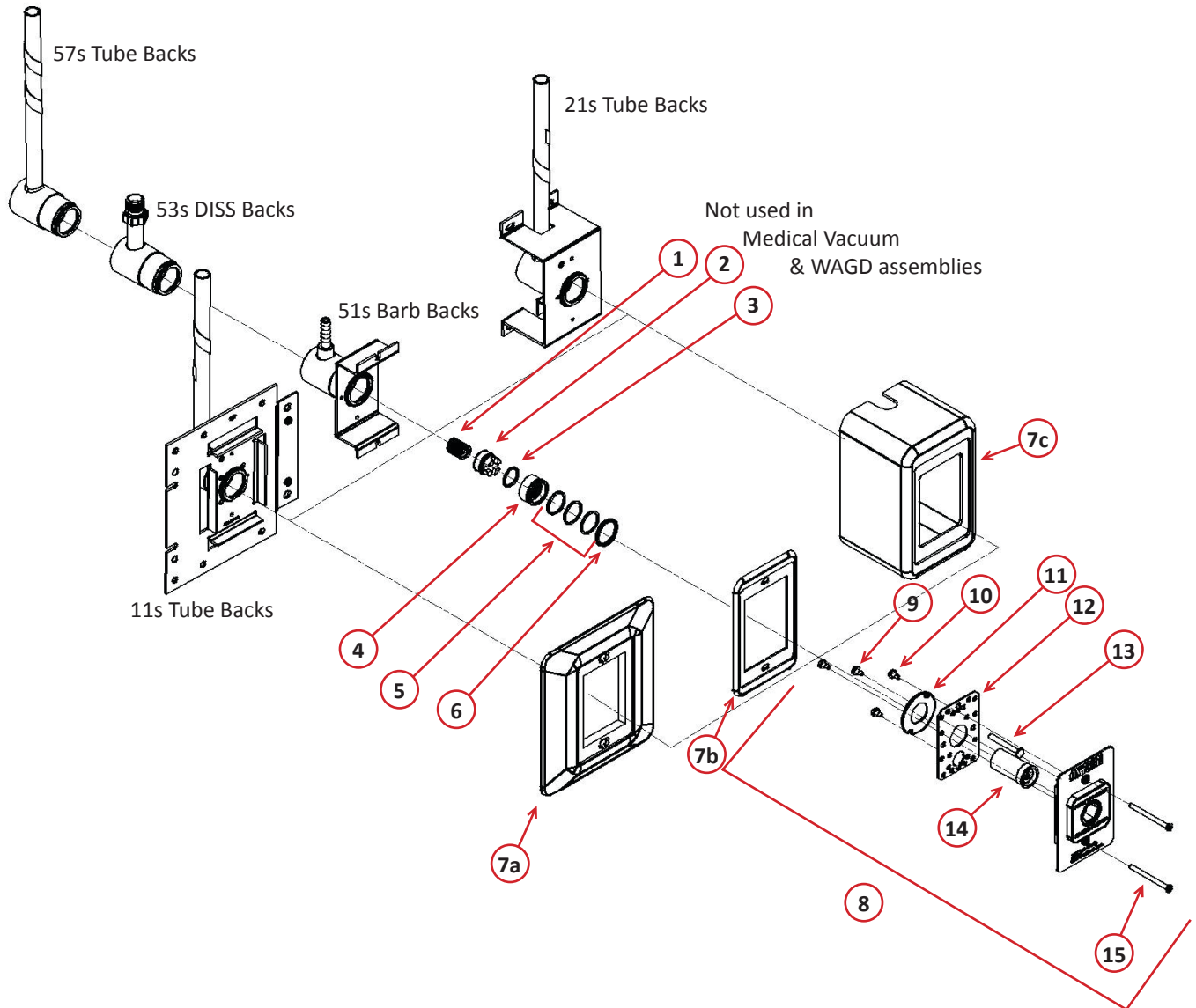

Item 8 includes items 9-15 in the Finishing Kit

Bill of Material

Item	Part Number	Description	Quantity	Note
1	PX-13-1003R	Spring	1	
2	PX-XC1000-5R	Poppet	1	
3	PX-11-0050R	Poppet O-Ring	1	A
4	PX-XC1000-6R	Insert Sleeve	1	
5	PX-11-0047R	Sleeve O-Ring	3	B
6	PX-12-1045R	Retaining Ring	1	
7a	PX-XC1000-9R	Finishing Kit Trim Plate	1	
7b	PX-XC1000-10R	Trim Plate Cover	1	
7c	PX-XC2100-3R	Finishing Kit Housing (Surface Mount)	1	
8	See Below	XP Finishing Kit Assemblies	See Below	
9	PX-12-3086R	Mounting Screws	2	
10	PX-12-3087R	Retaining Washer Mounting Screws	2	
11	PX-XP1000-14R	Retaining Washer	1	
12	PX-AQ1000-12R	Indexing Housing Cover	1	
13	PX-XP1000-1R	Indexing Pin	1	
14	PX-XP1000-ER	Front Body Assembly	1	
15	PX-30-0003	Mounting Screw Kit (2) PX-12-3037	1	
Item 8 Outlet Finishing Kits (all kits are assembled with items 9-14)				
	PX-XP1004-A	Nitrous Oxide	1	
	PX-XP1016-A	Medical Air	1	
	PX-XP1022-A	Medical Vacuum	1	
	PX-XP1024-A	Oxygen	1	
	PX-XP1032-A	WAGD (Waste Anesthetic Gas Disposal)	1	
Secondary Valve Kit				
	PX-XC1000KR	Secondary Valve Kit for all Gases Except Medical Vacuum & WAGD (Kit Includes Items 1 through 6)	1	
	PX-XC1022KR	Secondary Valve Kit for Medical Vacuum & WAGD (Kit Includes Items 4 through 6)	1	

NOTE: A – Install O-Ring item 3 on Poppet item 2
 B – Install O-Rings item 5 onto Insert Sleeve item 4