


**180° Puritan Bennett Style Outlets - PX-XP Series**

Oxygen Outlet Shown

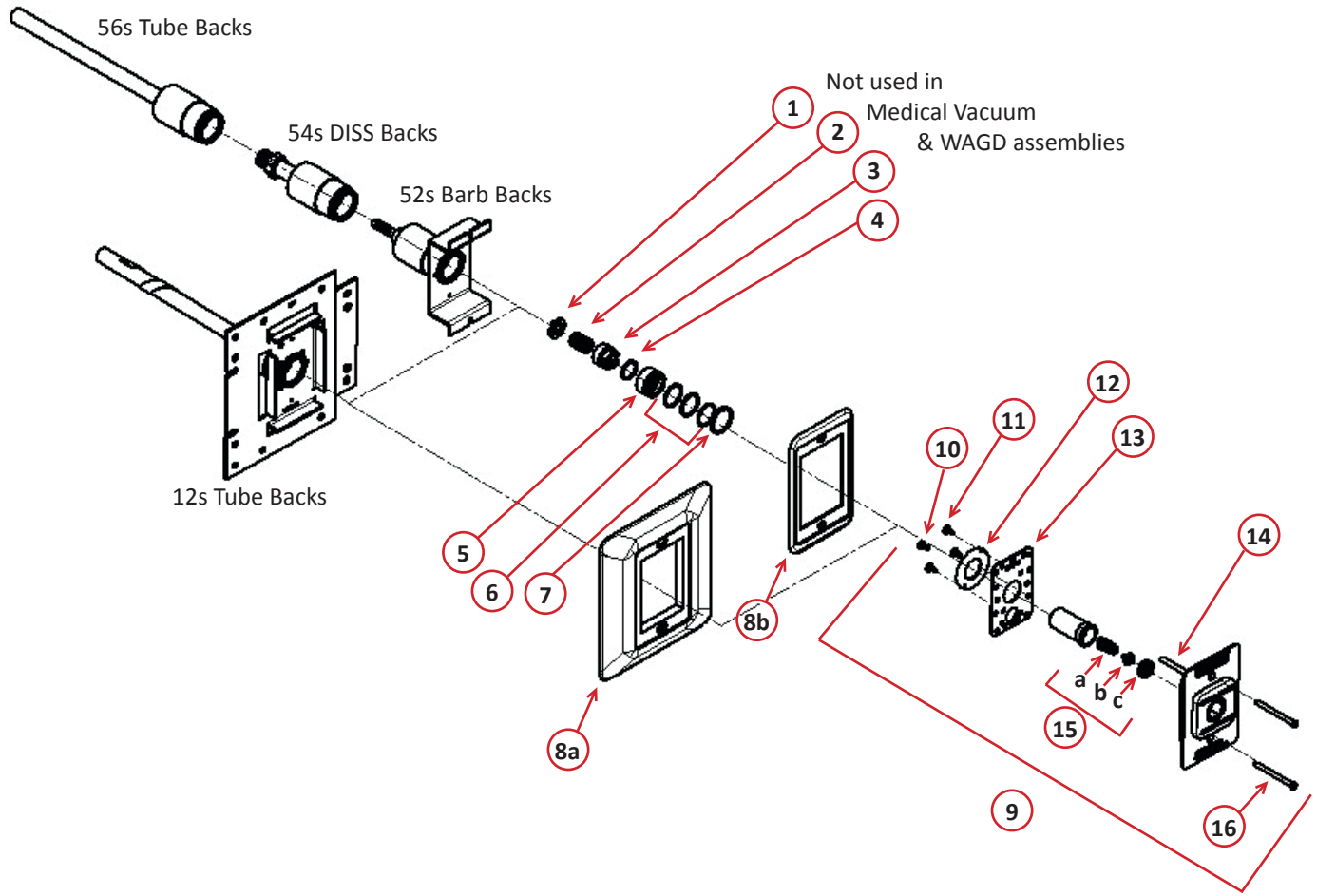

Item 9 includes items 10-16 in the Finishing Kit

**Bill of Material**

Item	Part Number	Description	Quantity	Note
1	PX-XC1000-7R	Spacer Washer	1	
2	PX-13-1003R	Spring	1	
3	PX-XC1000-5R	Poppet	1	
4	PX-11-0050R	Poppet O-Ring	1	A
5	PX-XC1000-6R	Insert Sleeve	1	
6	PX-11-0047R	Sleeve O-Ring	3	B
7	PX-12-1045R	Retaining Ring	1	
8a	PX-XC1000-9R	Finishing Kit Trim Plate	1	
8b	PX-XC1000-10R	Trim Plate Cover	1	
9	See Below	XP Finishing Kit Assemblies	See Below	
10	PX-12-3086R	Mounting Screws	2	
11	PX-12-3087R	Retaining Washer Mounting Screws	2	
12	PX-XP1000-14R	Retaining Washer	1	
13	PX-AQ1000-12R	Indexing Housing Cover	1	
14	PX-XP1000-1R	Indexing Pin	1	
15	PX-XP1000-ER	Front Body Assembly (See below for items included in the assembly)	1	
16	PX-30-0003	Mounting Screw Kit (2) PX-12-3037	1	
<b>Item 9 Outlet Finishing Kits</b> (all kits are assembled with items 10-16)				
	PX-XP1004-A	Nitrous Oxide	1	
	PX-XP1016-A	Medical Air	1	
	PX-XP1022-A	Medical Vacuum	1	
	PX-XP1024-A	Oxygen	1	
	PX-XP1032-A	WAGD (Waste Anesthetic Gas Disposal)	1	
<b>Item 15 PX-XP1000-ER Front Body Assembly</b> (kit is assembled with items a through c)				
a	PX-13-1016R	Poppet Spring	1	
b	PX-XP1000-10R	Poppet	1	
c	PX-XP1000-12R	Grommet Washer	1	
<b>Secondary Valve Kit</b>				
	PX-XC1000KR	Secondary Valve Kit for all Gases Except Medical Vacuum & WAGD ( Kit Includes Items 1 through 7 )	1	
	PX-XC1022KR	Secondary Valve Kit for Medical Vacuum & WAGD ( Kit Includes Items 5 through 7 )	1	

**NOTE:** A – Install O-Ring item 4 on Poppet item 3  
 B – Install O-Rings item 6 onto Insert Sleeve item 5